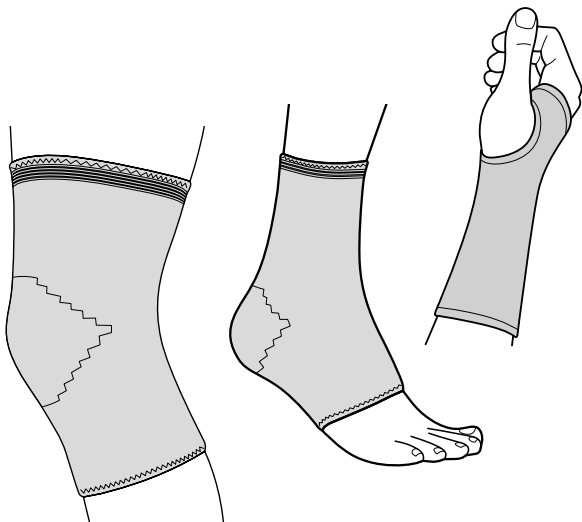
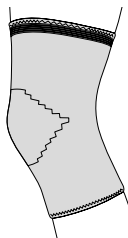
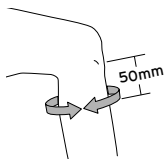
(de) Gebrauchsanweisung
(en) Instructions for use
(fr) Mode d'emploi
(it) Istruzioni per l'uso

(cs) Návod k použití
(pl) Instrukcja użytkowania
(sk) Návod na použitie
(hu) Használati útmutató

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Read these instructions carefully and keep them for future reference. If you give this product to someone else, remember to also include these instructions.

Intended use/Indication

The bandage is made using a skin-friendly material. It provides support, gentle stabilisation and relief in the case of slight instability and overexerted joints as well as for strains and sprains. It can be worn during physical activity and when exercising.

Consult your doctor if you have any questions.

Contra-indication

The bandage must **not** be used, except following consultation with a doctor, in the case of...

- ... external injuries
- ... circulatory disorders
- ... water retention (oedema)
- ... eczema or other skin disorders
- ... complex ligament instabilities
- ... complex localised pain syndromes.

Warning

- Keep the bandage and packing material out of the reach of infants. There is a risk of suffocation!

- Only use the bandage for its intended purpose. The bandage is designed for personal use and is not intended for commercial use.
- The bandage must not be worn continuously for more than 2 hours or for longer than 6 hours a day in total.
- If you experience discomfort when wearing the bandage, remove it immediately. The bandage must not restrict the flow of blood. The bandage must not be worn while asleep or by anyone who is unable to indicate that the bandage is uncomfortable.
- If a bandage is damaged, it must no longer be used.
- Only wear the bandage where the skin is clean and dry and do not apply any creams or ointments before putting it on.
- In some cases, particularly for those with allergies to certain materials (see sewn-in label), the bandage may cause discomfort or irritation of the skin.
- Serious cases must be reported to the manufacturer and the relevant authorities.
- **The bandage must be applied / positioned according to the illustrations provided on the front page of these instructions.**

Maintenance and storage:

- ▷ Wash the bandage before using it for the first time and at least once every second to third wearing. Follow the instructions on the sewn-in label.
- ▷ Dispose of the product and packaging in accordance with current environmental regulations.

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PS Product Services GmbH,